

Access the Menu

The **Menu**  will give you access to the Editor, Vocabulary Builder, and the Toolbox.

Use Word Finder

Use Word Finder to find a word and see where it is.

1. Select the keyboard icon to open the keyboard.
2. Select **Word Finder**.
3. Type the word and select **Search**.
4. The **Search Result** window will show one or more matching sequences for the word if available.
 - a. **Guide Me** will step you through the sequence by highlighting buttons in the sequence.
 - b. **Hide** will temporarily hide that sequence.
5. Select **Finished** to return to the home page.

Edit a Button

1. Select **Menu → Editor**.
2. Find the button you want to edit, then select **Edit**.
3. The “Edit” button will change color, then select the button you want to edit.

4. In the Edit Button Window:

- a. Change the **Button Message** to update what the button says.
- b. Change the **Button Label** to update what you see on the button.
- c. Change the **Image** for the button.
- d. Choose an image from **Suggested Symbols**.
- e. Choose **Camera** to take a photo.
- f. Choose the **Symbol Tab** on the left to search for more symbols.
- g. Select **Finished**, then select **Save**.

Create a New Page

1. Find the button you want to use to go to a new page, then select **Menu → Editor**.
2. Select **Edit** and choose the button you want to edit. Add **Label** and **Symbol**.
3. Select the **Navigation** tab → **Go to Page** → **Add New** → **Name Page** → **Create** → **Finished**.
4. Select the button to open the new page and continue by editing buttons on the new page.

Menu Lock

Use Restrictions to lock the menu button via a password or a USB flash drive.

Lock/Unlock with Password

1. Select **Menu → Toolbox → User Settings → Restrictions**.
2. Select **Password** and follow the prompts to create a password.
3. To unlock, select **Menu**, enter the password to unlock the menu.

Lock/Unlock with USB flash drive

1. Select **Menu → Toolbox → User Settings → Restrictions**.
2. Select **USB**, then insert a USB flash drive and follow the prompts.
3. Remove the USB flash drive and the Menu button will be locked.
4. To unlock, insert the USB flash drive and select **Menu**.

Change the Voice

1. Select **Menu → Toolbox → User Settings → Voice**.
2. Choose **Main Voice** or **Prompt Voice**.
3. Select **Change Voice**, then choose **Adult** or **Youth**.
4. Try out voices, then select **Choose** for the voice you want.
5. Select **Finished**.

Change the Way the Device Pronounces a Word

1. Select **Menu → Toolbox → User Settings → Voice**.
2. Choose **Main Voice** or **Prompt Voice**.
3. Select **Edit Pronunciation**.
4. Type the word in the **Enter Word** box.
5. Type how the word sounds in the **Enter Phonetic Spelling** box.
6. Select **Finished** to close the keyboard.
7. Select the **play button** ▶ to test it.
8. When it sounds right, select **Add** then select **Close**.
9. Select **Finished**.

Vocabulary Builder

Use Vocabulary Builder to show fewer words for a short time.

1. Select **Menu → Vocabulary Builder**.
2. Turn the **Off/On** toggle on. 
3. Select **Edit List**.
4. Type a word or multiple words separated by commas.
5. Select **Finished**. A message will confirm that the word(s) were added.

6. Select **Okay** to close the message. The words will be listed below.
7. Select **Finished**, then select **Exit Menu**.
8. When you are done, select **Menu → Vocabulary Builder**.
9. Turn the **Off/On** toggle off.

Update Empower Software Using Wi-Fi

If you see a purple exclamation point, it means a software update is available.

1. Select **Menu → Toolbox → System Settings → Updates**.
2. Select **Check for Updates** next to Update Using Wi-Fi. If an update is found, an Update Available window will open.
3. Select **Download**. The update may take a long time. Plug in the charger and do not interrupt the update.
4. Follow the on-screen prompts.
5. Select **Install**.
6. A message will show when the update is done.

Back Up a Vocabulary File to a USB Drive

1. Connect the USB drive to the Accent.
2. Select **Menu → Toolbox → System Settings**.

3. Select **Export & Import → Export → Choose Files**.
4. Select **Export** to the right of the file you want to back up.
5. Navigate to the USB drive. If needed, select the **up one level button**  to find the USB drive.
6. Select **Finished** then **Save**.
7. When the backup is complete, an “Export Succeeded” message will appear.
8. Remove the USB drive and keep it in a safe place.

Find Video Tutorials

[PRC-Salttillo YouTube Channel](#) → **Accent Speech Generating Devices** section → **Accent with Empower Playlists**

